

### Zero Adjusting the FUTREX-5500

Prior to beginning body fat measurements, the FUTREX-5500 must be Zero Adjusted:

- ? Insert the Light Wand into the Optical Standard as shown in the Illustration.
- ? Be sure that the silver stripe on the Light Wand is aligned with the silver stripe on the Optical Standard.

***Important note: The Light Shield must not be on the Light Wand when Zero Adjusting the FUTREX-5500.***

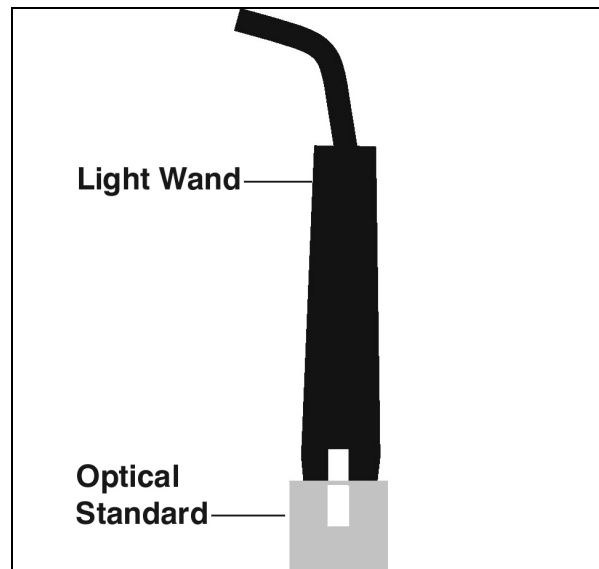


Illustration 1

Press the "ZERO ADJUST" Button on the keypad.

The display will now read "READY."

You may proceed with the body fat measurement.