

Proper Placement of the Light Wand

Place the Light Shield on the Light Wand as shown in Illustration 3, aligning the white stripe on the Light Shield with the silver stripe on the Light Wand. Be sure that the Light Wand protrudes approximately 1/8" (3 mm) past the bottom of the Light Shield.

Note: The wand shown in the FUTREX-5500 wand, however, the same technique is used for the FUTREX-6100.

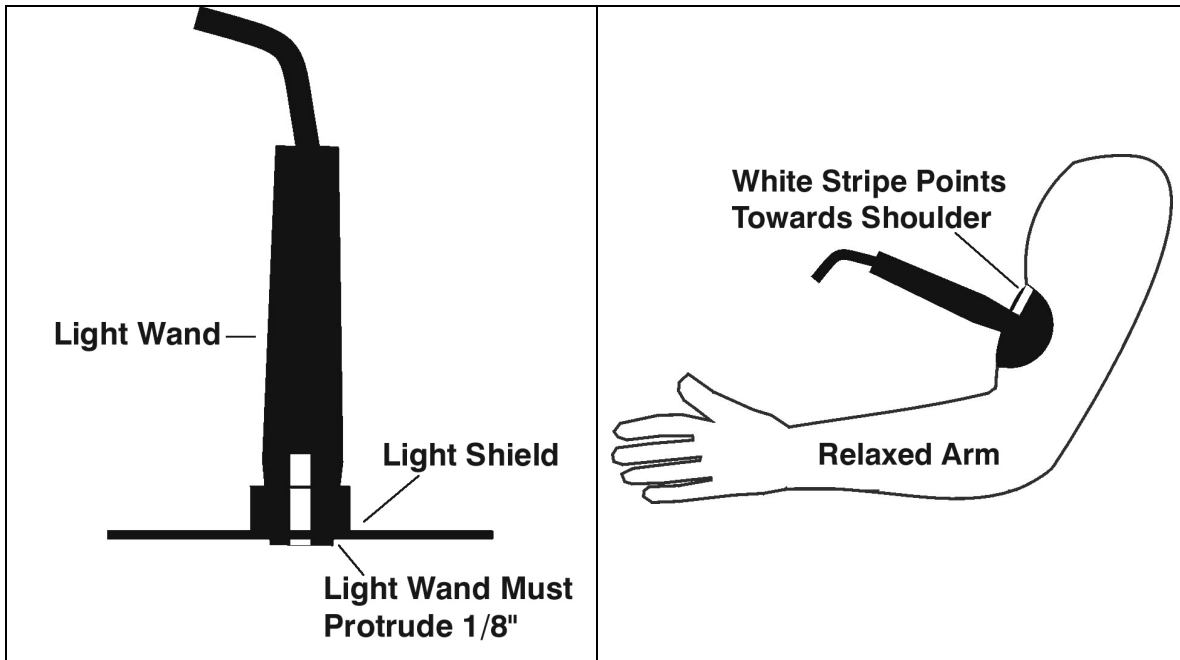



Illustration 3

Illustration 4

Place the center of the Light Wand on the Measurement Site (the procedure for locating the measurement site is described under the heading: *Selecting The Measurement Site*). Be sure that the white stripe is pointing directly towards the shoulder as shown in Illustration 4. The arm should be resting on a table top with the elbow bent as shown in Illustration 4. The arm must be relaxed during the measurement. *Note: Always use the Light Shield to block external light.*

Press the push button on the Light Wand. The following message will be displayed: "Remove and replace."

Remove the Light Wand by gently rocking the wand off of the arm and then replace it on the same site as shown in the Illustrations 5 & 6. Then again, press the push button on the Light Wand.

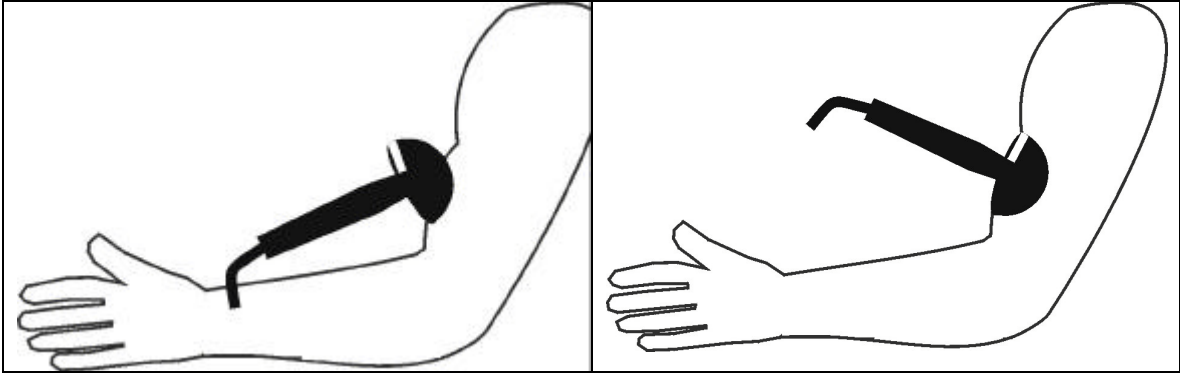

Remove
Illustration 5

Replace
Illustration 6